

Adhesive Remover



Product Features:

- Quickly and effectively removes stickers, labels, tape residue, adhesives, oil, grease, wax, glue, etc
- Neutral pH formula
- Safe for use on a variety of metals, glass, plastic and rubber
- Biodegradable
- Fragrance-free

Key Formula Ingredients:

- Soy based
- Unique surfactant blend
- Corrosion inhibitor

Key Chemical Attributes

pH Neat	6.5-8.5
pH Use Dilution	6.5-8.5
Density	0.930-0.935
Color	Light Yellow

Handling and Storage:

Do not store in extreme hot or cold temperatures.

Adhesive Remover

For the removal of tape and other adhesive residue from the surface of surgical instrumentation, case carts, stainless steel, etc.

Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.


Instructions for Use - Label/Tape Removal:

1. Apply adhesive remover liberally on label or tape and around edges
2. Allow to soak for 3-5 minutes
3. Lift corners and peel back slowly

Instructions for Use - Residue Removal:

1. Apply adhesive remover liberally
2. Allow to sit for 2 minutes
3. Wipe clean with dry cloth
4. Rinse thoroughly or follow normal cleaning process

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Adhesive Remover		106-8093	2182034	8 oz	12



For more information, visit belimed.com

REV12.22.2020_Q0

Engineers of Confidence.

Belimed
Infection Control

Belimed, Inc.
1.800.451.4118 | 1.843.216.7424
belimed.com