

Descaler/Delimer



Product Features:

- Concentrated acid formula for the fast acting removal of scale and lime from stainless steel surfaces
- Improves equipment performance and appearance
- Prevents future scale and lime build up on equipment

Key Formula Ingredients:

- Phosphoric acid
- Unique surfactant blend

Key Chemical Attributes

pH Neat	0.6-1.5
pH 1% Dilution	1.8-2.3
Density	1.08-1.09
Color	Light Straw

Handling and Storage:

Use with adequate ventilation. Avoid breathing vapor spray or mist. Store in a cool dry place. Keep partially used containers closed and away from heat and humidity. Do not expose to extreme hot or cold temperatures. Keep out of reach of children. Keep in properly labeled containers.

Acidic cleaning solution for removal of lime and scale on equipment – Concentrate

For cleaning inside sterile processing equipment and stainless steel surfaces.

Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.



DANGER: Corrosive acid pH<2. Will cause eye damage, intestinal damage, harmful to skin. Wash hands and exposed areas thoroughly after handling.




Wear eye protection and protective gloves.

Instructions for Use:

1. For light deposits and preventative maintenance create a solution of 1-3 oz. per gallon of water
2. For moderate deposits create a solution of 4-8 oz. per gallon of water
3. For heavy deposits create a solution of up to 32 oz. per gallon of water
4. Follow facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Descaler/ Delimer		106-8092	2181034	1 gal	4
		110-1528	2181035	5 gal	1



For more information, visit belimed.com

REV12.22.2020_Q0

Engineers of Confidence.

Belimed
Infection Control

Belimed, Inc.
1.800.451.4118 | 1.843.216.7424
belimed.com