

Enzymatic Detergent Ultrasonic Cleaning



Specially formulated multi-enzyme detergent designed to optimize the cleaning process in ultrasonic equipment – Concentrate

For ultrasonic cleaning of surgical grade instruments/reusable items, endoscopes and other devices.

Product Features:

- Proprietary, subtilisin-free formulation
- pH neutral
- Non-corrosive
- Low foaming
- Gentle on instruments
- Will not interfere with ultrasonic cavitation
- Biodegradable and aquatic life safe

Key Formula Ingredients:

- Three (3) Enzymes (Protease for proteins, lipase for fatty acids and amylase for starches)
- Unique blend of surfactants for cleaning efficacy and preventing bioburden from redepositing on instruments
- Corrosion inhibitor
- Sequestering agent for neutralizing hard water minerals

Key Chemical Attributes

pH Neat	8.0-9.0
pH Use Dilution	6-8
Density	1.050-1.060
Color	Blue

Material Compatibility:

The Belimed Protect® Enzymatic Detergent for Ultrasonic Cleaning demonstrates a wide range of material compatibility and is suitable for the following materials:

- Stainless Steel
- Aluminum
- Anodized Aluminum
- Titanium
- Copper
- Brass
- All commonly used plastics and elastomers, including Polyurethane, Silicone, and Teflon
- Latexes and Synthetics (Vinyl and Nitrile)

Handling and Storage:

Do not store at extreme hot or cold temperatures.

Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.

Instructions for Use:

1. Add 1/12 to 1/2 oz per gallon (0.7 to 4 ml/L) of water
2. Set the water temperature and equipment cycle per facility policies and procedures
3. Drain, fill, and redose ultrasonic per facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Enzymatic Detergent Ultrasonic Cleaning Concentrate	[Pink Box]	106-8036	2130001	1 gal	4
		106-8038	2130002	2.5 gal	2
		106-8039	2130005	5 gal	1
		106-8040	2130015	15 gal	1
Enzymatic Detergent Ultrasonic Cleaning Concentrate with Fragrance	[Pink Box]	107-4540	2131001	1 gal	4
		107-4541	2131002	2.5 gal	2
		107-4543	2131005	5 gal	1

REV12.22.2020_Q0

Engineers of Confidence.