

Enzymatic Detergent Washer Disinfectant



Specially formulated, multi-enzyme detergent designed to optimize the cleaning process in automated washer disinfectors – Concentrate

For automated cleaning of surgical instruments, other reusable items, and devices.

Product Features:

- pH neutral
- Non-corrosive
- Gentle on instruments
- Fragrance free
- Biodegradable and aquatic life safe

Key Formula Ingredients:

- Three (3) Enzymes (Protease for proteins, lipase for fatty acids and amylase for starches)
- Unique blend of surfactants solubilize bioburden
- Corrosion inhibitor

Key Chemical Attributes	
pH Neat	8.0-9.0
pH 1% Dilution	6.0-8.0
Density	1.050-1.060
Color	Blue

Material Compatibility:

The Belimed Protect® Enzymatic Detergent WD demonstrates a wide range of material compatibility and is suitable for the following materials:

- Stainless Steel
- Aluminum
- Anodized Aluminum
- Titanium
- Copper
- Brass
- All commonly used plastics and elastomers, including Polyurethane, Silicone, and Teflon
- Latexes and Synthetics (Vinyl and Nitrile)

Handling and Storage:

Do not store at extreme hot or cold temperatures.

Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.

Instructions for Use:

1. Automatically feed 1/8 to 3/4 oz of detergent per gallon (1 to 6 ml/L) of water
2. For best results, water temperature should be 85°F - 150°F (30°C - 65°C)
3. Follow facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Enzymatic Detergent WD Concentrate, Low Foam		106-8046	2140001	1 gal	4
		106-8048	2140002	2.5 gal	2
		106-8049	2140005	5 gal	1
		106-8050	2140015	15 gal	1
		106-8051	2140030	30 gal	1
		106-8052	2145055	55 gal	1



For more information, visit belimed.com

REV12.22.2020_Q0

Engineers of Confidence.