

Lubricant WD



Instrument lubricant formulated for automated washer disinfectors – Concentrate

Designed for automated washing of surgical instruments, other reusable items, and devices.

Product Features:

- Special formulation designed to lubricate and protect instruments, hinges, box locks and general equipment
- pH neutral
- Gentle on instruments
- Fragrance free
- Biodegradable and aquatic life safe

Key Formula Ingredients:

- Food-grade mineral oil
- Unique blend of surfactants for improved cleaning efficacy

Key Chemical Attributes	
pH Neat	5.5-7.5
pH 1% Dilution	6.0-7.0
Density	0.985-1.005
Color	Milky White

Material Compatibility:

The Belimed Protect® Lubricant WD demonstrates wide range of material compatibility and is suitable for the following materials:

- Stainless Steel
- Aluminum
- Anodized Aluminum
- Titanium
- Copper
- Brass
- All commonly used plastics and elastomers, including Polyurethane, Silicone, and Teflon
- Latexes and Synthetics (Vinyl and Nitrile)

Handling and Storage:

Do not store at extreme hot or cold temperatures.


Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label. Always refer to instrument IFU regarding lubricant use.

Instructions for Use:

1. Automatically feed 1/12 to 1/3 oz of lubricant per gallon (0.7 to 2 ml/L) of water to the washer disinfectant
2. Follow facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Lubricant WD		106-8056	2150001	1 gal	4
		106-8058	2150002	2.5 gal	2
		106-8059	2150005	5 gal	1
		106-8060	2150015	15 gal	1
		106-8051	2150030	30 gal	1
		106-8062	2150055	55 gal	1



For more information, visit belimed.com

REV12.22.2020_Q0

Engineers of Confidence.