

Neutral Non-Enzymatic Detergent



Neutral Non-Enzymatic Detergent for cleaning case carts and instrument containers in Cart Washers and Washer Disinfectors

For manual or automated cleaning of surgical instruments, other reusable items and devices.

Product Features:

- pH neutral and non-corrosive
- Low foam concentrate formulation
- Gentle ingredients will not damage carts or casters
- Fragrance free
- Biodegradable

Key Formula Ingredients:

- Surfactants for surface wetting and cleaning efficacy
- Corrosion inhibitor

Key Chemical Attributes	
pH Neat	6.5-8.0
pH 1% Dilution	7.0-7.5
Density	1.020-1.030
Color	Straw

Material Compatibility:

The Belimed Protect® Neutral Non-Enzymatic Detergent demonstrates wide range of material compatibility and is suitable for the following materials:

- Stainless Steel
- Aluminum
- Anodized Aluminum
- Titanium
- Copper
- Brass
- All commonly used plastics and elastomers, including Polyurethane, Silicone, and Teflon
- Latexes and Synthetics (Vinyl and Nitrile)

Handling and Storage:

Do not store at extreme hot or cold temperatures.


Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.

Instructions for Use:

1. Add 1/8 to 1/2 oz of detergent per gallon (1 to 4 ml/L) of water
2. Follow facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Neutral Non-Enzymatic Detergent		106-8067	2160001	1 gal	4
		106-8069	2160002	2.5 gal	2
		106-8070	2160005	5 gal	1
		106-8071	2160015	15 gal	1
		106-8072	2160030	30 gal	1
		106-8073	2160055	55 gal	1



For more information, visit belimed.com

Engineers of Confidence.