

Neutral Non-Enzymatic Pretreatment Spray



Dual foam/liquid pretreatment spray specially formulated for ophthalmic instrumentation

For pre-cleaning and transportation of surgical grade instruments, reusable items, scopes and other devices.

Product Features:

- Safe for use on ophthalmic instruments
- Non-irritating to both skin and eyes
- Easily rinsed
- Fragrance-free
- Stays wet up to 72 hours
- Safe for end users

Key Formula Ingredients:

- Unique blend of surfactants solubilize bioburden and retain wetness
- Corrosion inhibitor

Key Chemical Attributes

pH Neat	6.5-8.5
pH 1% Dilution	N/A
Density	1.010-1.020
Color	Light Straw

Handling and Storage:

Do not store at extreme hot or cold temperatures.

Safety Information:

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.


Recommended Practice:

- After each case, staff should spray all instrument trays
- The instruments should be covered completely with product
- After a few hours, check trays and re-spray if necessary

Instructions for Use:

1. Shake bottle before each use
2. Spray instruments and equipment as soon as possible after procedure
3. Keep instruments and equipment wetted with solution until decontamination process is started
4. Follow facility policies and procedures

Product Ordering Information:

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Neutral Non-Enzymatic Pretreatment Spray		1032459	2110016	16 oz	12



For more information, visit belimed.com

REV12.22.2020_Q0

Engineers of Confidence.

Belimed
Infection Control

Belimed, Inc.
1.800.451.4118 | 1.843.216.7424
belimed.com