

## Product Specification

# SmartHub Orbit

### Product Description

Belimed's SmartHub Orbit is an easy to use web application that allows the user to access CSSD equipment information remotely. SmartHub Orbit gathers data from every Belimed machine in the sterile processing department and turns it into useful, well visualized information for the staff. SmartHub Orbit is an annual subscription service available in the following tiers.


### Available Now

- **Starter Pack** (complimentary upon migration from ICS)
- **Core**
- **Advanced**

NOTE: SmartHub Orbit does not process any patient or operator data.


### Features

- Unlimited User Accounts
- Remote 24/7 Access to Information on any Mobile Device via Web Application
- Real-time Remote Access to Machine Status
- Comprehensive Records of Batches
- Digitalized Versions of Latest Available Machine User Manuals
- Alerts and Notifications
- Explanation of Error Codes
- Troubleshooting Assistance with Video Guidance (Advanced Only)
- Access to Multiple SPD Sites via Single Login
- Archive of Alarm History
- E-learning Modules for Equipment (Advanced Only)
- Searchable Filters for Past Batch Records, Errors
- Advanced Analytics Features (Advanced Only)
- Service History with Reports (Advanced Only)

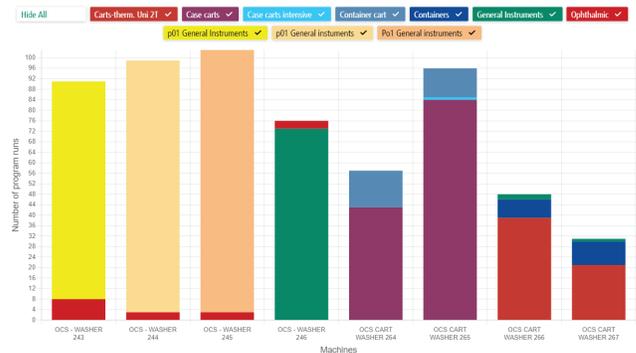


### Network Topology - Cloud Application

#### Prerequisite: SmartHub Connect (\*On-site Application)

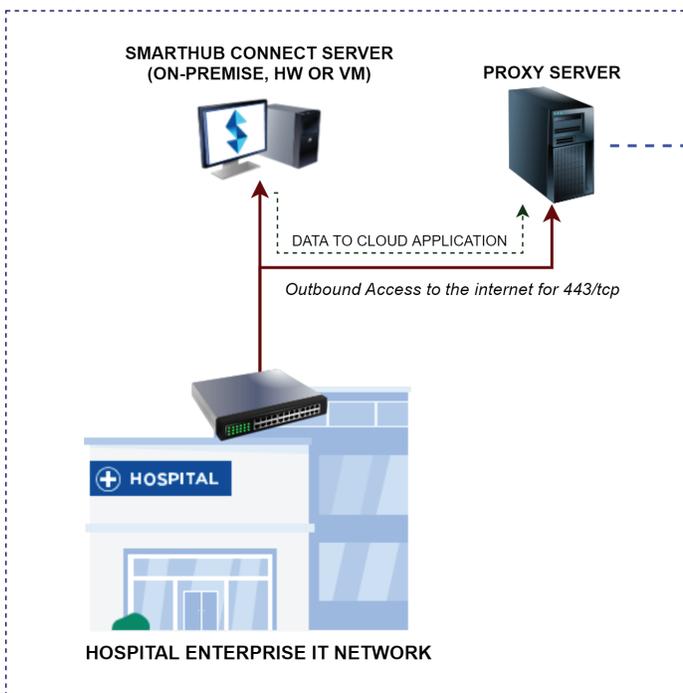

#### SMARTHUB ORBIT CLOUD APPLICATION

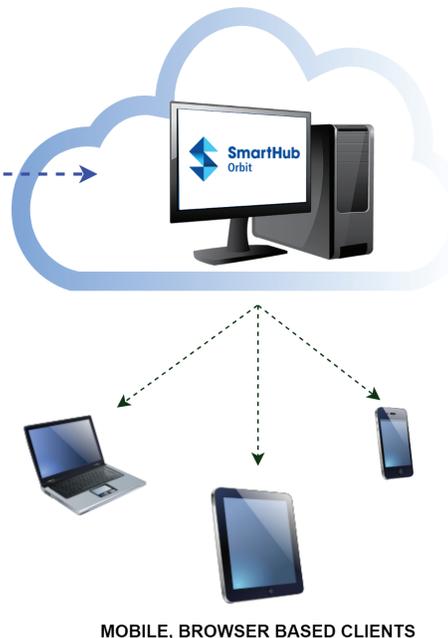

\*refer to SmartHub Connect specification sheet

## Product Specification

# SmartHub Orbit

FEATURES	STARTER	CORE	ADVANCED
Unlimited Number of Users	✓	✓	✓
24/7 Information Access (Mobile & Desktop)	✓	✓	✓
Machine Status Remote Access in Real Time	✓	✓	✓
Batch Records	✓	✓	✓
Digitalized Machine Manuals	✓	✓	✓
Alarms / Notifications	✓	✓	✓
Description of Error Codes	✓	✓	✓
Multi-site Capability	✓	✓	✓
Alarm History	✓	✓	✓
E-learning Equipment Modules	✓	—	✓
Troubleshooting Videos	✓	—	✓
Filter Using Search Criteria on Past Batch Records	✓	—	✓
Analytics	✓	—	✓

### Ordering

Part Number	Description
1066684	SmartHub Orbit – Starter Subscription, 1 year (Conversion ICS to SH customers)
1064504	SmartHub Orbit – Core Subscription, 1 Year
1064505	SmartHub Orbit – Advanced Subscription, 1 Year



### Network Environment Requisites

Feature	System Requirement									
<b>Prerequisite</b>	Belimed Equipment must be configured and paired with a running on-site SmartHub Connect application as well as open ports to allow internet communication.									
<b>Internet Access</b>	443/tcp									
<b>Option 1: Direct Access IP Range for Outbound</b>	[“13.69.71.0/25”, “13.69.109.0/25”, “13.95.15.251/32”, “23.100.4.253/32”, “23.100.8.130/32”, “40.113.153.50/32”, “40.113.176.160/27”, “40.113.176.192/27”, “40.113.177.0/24”, “40.118.27.192/32”, “51.144.118.31/32”, “52.236.189.128/25”, “2603:1020:206:402::300/123”, “2603:1020:206:802::240/123”, “2603:1020:206:c02::240/123”]									
<b>Option 2 : Via proxy Server</b>	<table border="1"> <thead> <tr> <th>Environment</th> <th>IoTHub</th> <th>Storage</th> </tr> </thead> <tbody> <tr> <td>PROD</td> <td>iot-prod-sho-be-1.azure-devices.net</td> <td>stblobprodshobe1.blob.core.windows.net</td> </tr> <tr> <td>TEST</td> <td>iot-test-sho-be-1.azure-devices.net</td> <td>stblobtestshobe1.blob.core.windows.net</td> </tr> </tbody> </table>	Environment	IoTHub	Storage	PROD	iot-prod-sho-be-1.azure-devices.net	stblobprodshobe1.blob.core.windows.net	TEST	iot-test-sho-be-1.azure-devices.net	stblobtestshobe1.blob.core.windows.net
Environment	IoTHub	Storage								
PROD	iot-prod-sho-be-1.azure-devices.net	stblobprodshobe1.blob.core.windows.net								
TEST	iot-test-sho-be-1.azure-devices.net	stblobtestshobe1.blob.core.windows.net								

REV10.25.2023\_Q0

Engineers of Confidence.

**Belimed**  
Infection Control

Belimed, Inc.  
1.800.451.4118 | 1.843.216.7424  
[belimed.com](http://belimed.com)