

# BRIGHT RDA

Order details: 5 / 10 / 20 / 210 / 1,000 litres



## A neutral rinse aid and drying aid in washer-disinfectors

Spot- and streak-free drying of surgical instruments, bedsteads, carts and trolleys.

Belimed Protect™ BRIGHT RDA is formulated especially for use in automated processes, for spot-free and streak-free rinsing and drying. Suitable for all water qualities, this surfactant-based formula reduces water tension and therefore significantly shortens drying time for cart washers, washer-disinfectors for surgical instruments and associated equipment.

Drying time is reduced by dosing this product into the last hot rinse, where it reduces surface tension on the instruments by allowing water to slide off surfaces as a film, without leaving drops.

Belimed Protect™ BRIGHT RDA is also designed to prevent the formation of lime deposits on process parts and in washing machine components (e.g. nozzles, pipes, etc.) and to prevent the formation of water marks.

Belimed Protect™ BRIGHT RDA is registered as a Class I medical device.

### ITS PRIMARY FEATURES

- Improves drying efficacy – spot- and streak-free
- Reduces drying time – increases throughput
- pH-neutral – broad material compatibility
- Helps prevent scale
- Non-foaming
- Prevents the build-up of water scale in automated washer equipment
- Gives a smooth and shiny appearance to processed instruments
- Prevents pitting from corrosion
- Prevents the formation of rust on instruments

### Applications

Belimed Protect™ BRIGHT RDA is ideal as a neutral drying booster in washer-disinfector equipment.

Belimed Protect™ BRIGHT RDA is not suitable for manual applications.

### Directions for use

Please follow the manufacturer’s recommendations for the reprocessing of medical devices according to EN ISO 17664. For more details regarding instrument reprocessing, please visit [www.a-k-i.org](http://www.a-k-i.org).

**Note:** Excessive doses may cause the formation of white residues on processed instruments and utensils.

Product Description	Belimed Protect™ BRIGHT RDA
Dilution	0.5–1 ml/l
CE Mark	Fulfils the requirements of European Regulation (EU) 2017/745
Appearance	Transparent, colourless to pale straw liquid
pH	7.5 (neat solution)
Storage conditions	5–25 °C
Shelf life	36 months
Disposal instructions	See Safety Data Sheet

## Product properties overview

This table is for guidance only.

	Intended use								Microbicidal spectrum							pH value*		
	Medical device cleaning	Medical device disinfection	Medical device maintenance/care	Bedpan machine cleaning	For glassware	For thermolabile instruments	For flexible endoscopes	Manual reprocessing of medical devices	Bactericidal (all vegetative bacteria incl. multi-resistant bacteria, e.g. MRSA, VRE or ESBL)	Levurocidal (yeasts)	Fungicidal (fungi)	Mycobactericidal (e.g. tuberculosis bacteria)	Virucidal for enveloped viruses (incl. HBV, HCV and HIV)	Fully virucidal (enveloped and nonenveloped viruses, e.g. papillomavirus or parvovirus)	Mild alkaline	Neutral	Acidic	
<b>Belimed Protect™ CLEAN</b>																		
CLEAN MAE	•				•	•	•	•						•				
CLEAN NE	•				•	•	•	•							•			
CLEAN MA	•				•	•								•				
CLEAN Pre Treat	•		•		•									•				
<b>Belimed Protect™ SAFE</b>																		
Glutaraldehyde Disinfectant		•				•	•	•	•	•	•	•	•		•			
Cleaner and Disinfectant	•	•						•	•		•	•	•		•			
PAA 15		•				•	•	•	•	•	•	•	•				•	
<b>Belimed Protect™ NEUTRA</b>																		
NEUTRA-1			•		•												•	
<b>Belimed Protect™ BRIGHT</b>																		
BRIGHT RDA			•		•										•			
BRIGHT BP				•													•	
<b>Belimed Protect™ CARE</b>																		
CARE Lubricant WD			•												•			
CARE SR			•														•	

\*Ready-to-use solution

## Disposal of spent solutions

Flush to drain with copious water or soak up onto an inert material and dispose of with clinical waste. Product is biodegradable under OECD conditions.

## Health and safety




- Belimed Protect™ BRIGHT RDA is an irritant to eyes and skin. It is fully biodegradable.
- Belimed Protect™ BRIGHT RDA is intended for machine use only. Contact with skin and eyes should be avoided.
- See Safety Data Sheets for details.

## Associated products and services

The Belimed Protect™ range of chemicals has been developed and validated with Belimed washer-disinfectors and AERs to meet the cleaning efficacy requirements of medical devices according to the consistent and repeatable standards outlined in EN ISO 15883. For further support regarding the optimization of your cleaning process and/or troubleshooting, please contact your local Belimed representative.

## SYMBOLS AND COLOUR CODING SYSTEM

### Symbols – recommended use

-  For automated reprocessing in WDs and AERs
-  For manual reprocessing
-  Safe for use with flexible endoscopes

### Colour coding system – product category

-  For cleaning
-  For rinsing and drying
-  For disinfecting
-  For maintenance and care
-  For neutralizing



Amity Ltd., Libra House, West Street,  
Worsbrough Dale, Barnsley, South Yorkshire, S70 5PG, UK



Amity Healthcare (EU), Ltd Unit 24–25,  
Grattan Business Park Clonsaugh, D17 TK25 Dublin, Ireland



Belimed GmbH, Edisonstraße 7a,  
84453 Mühldorf am Inn, Germany



1639

[www.belimed.com](http://www.belimed.com)

**Belimed**  
Infection Control