




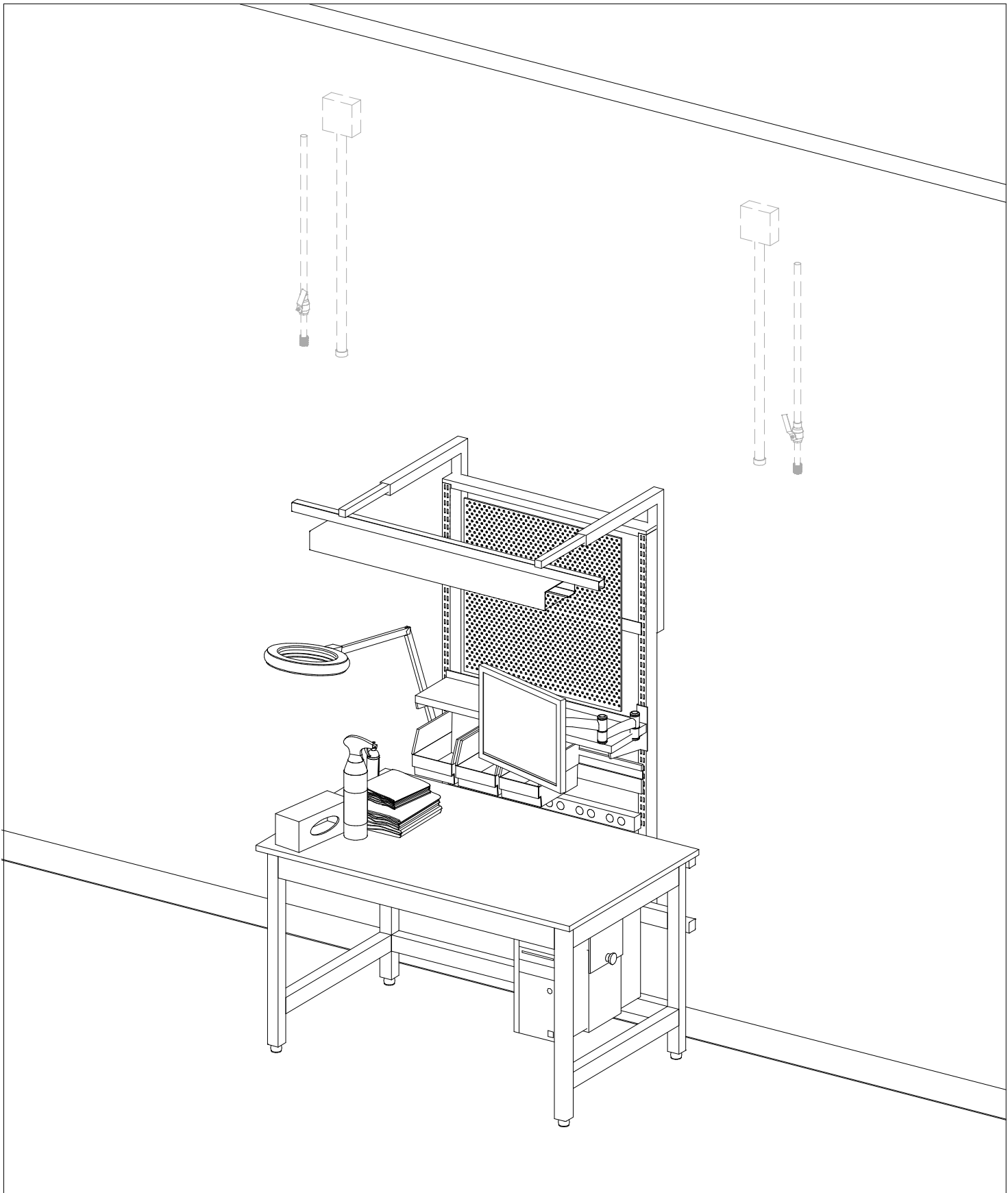


Belimed Infection Control 8351 PALMETTO COMMERCE PKWY SUITE 101 LADSON, SC 29456 TEL: (800) 457-4117 WWW.BELIMED.COM	TDS-P&PW	BELIMED PREP & PACK WORKSTATION	DRAWN BY: A. GRIFFIN	
	REV: E		SCALE: NONE	
	3D VIEW	TYPICAL INSTALLATION OF PREP & PACK WORKSTATION	DATE: 10/27/2025	PAGE: 01 OF 05

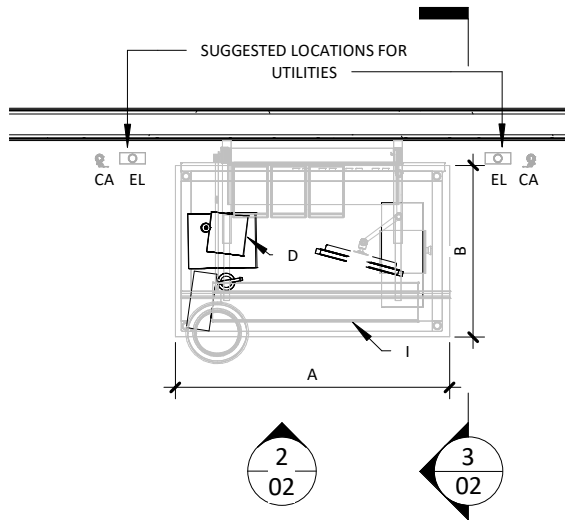


1 UTILITY LAYOUT
3/8" = 1'-0"

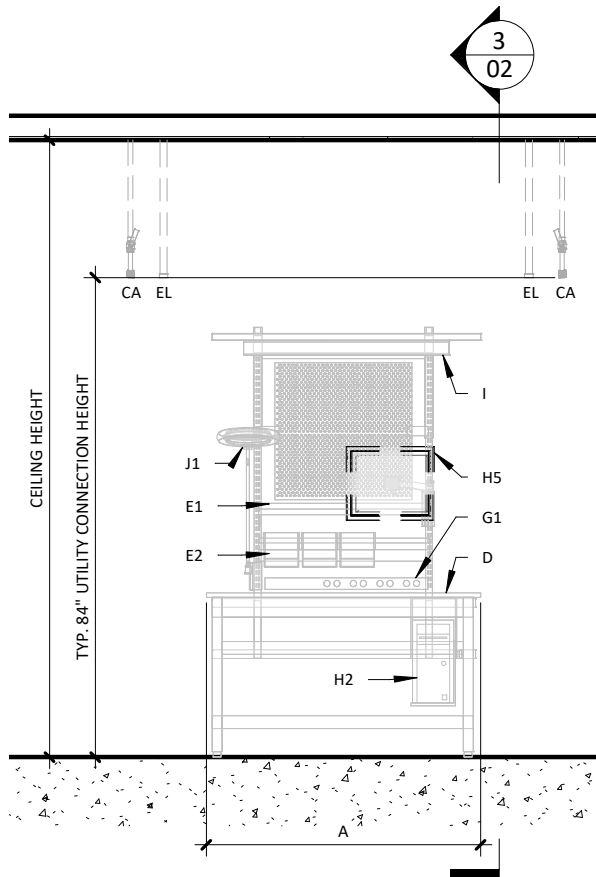


2 UTILITY FRONT ELEVATION
3/8" = 1'-0"

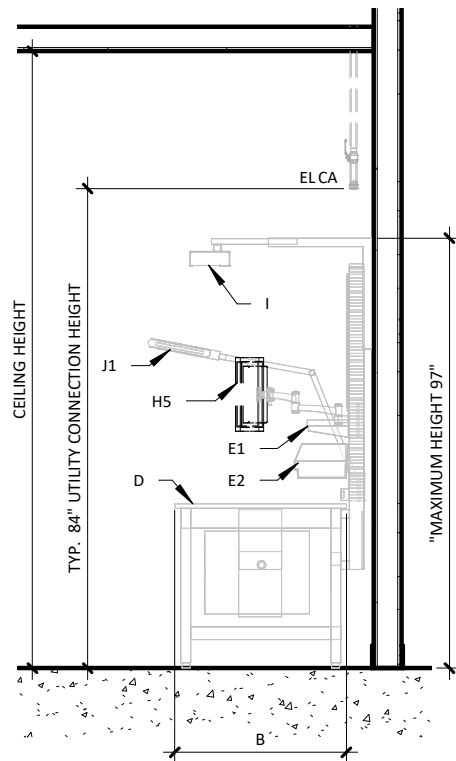


3 UTILITY SIDE ELEVATION
3/8" = 1'-0"

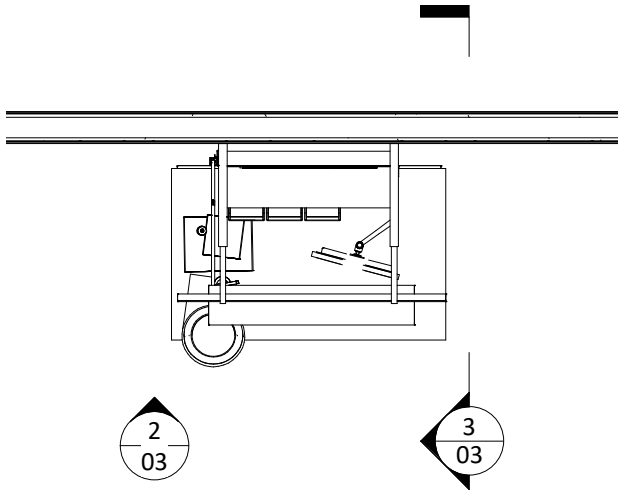

1 EQUIPMENT PLACEMENT PLAN
3/8" = 1'-0"

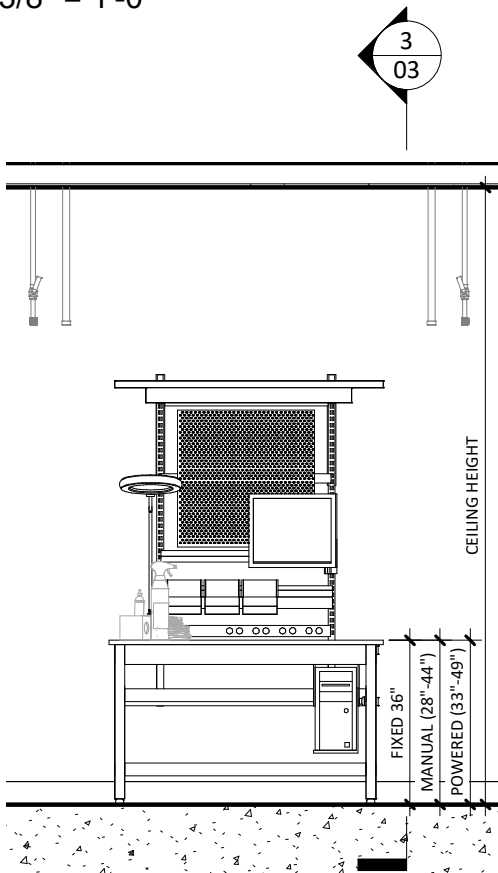

2 EQUIPMENT FRONT ELEVATION
3/8" = 1'-0"

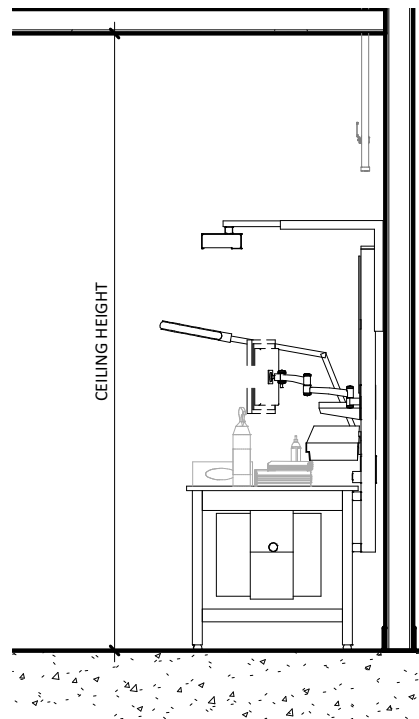

3 EQUIPMENT SIDE ELEVATION
3/8" = 1'-0"

Belimed
Infection Control

8351 PALMETTO COMMERCE PKWY SUITE 101
LADSON, SC 29456
TEL: (800) 457-4117 WWW.BELIMED.COM

TDS-P&PW

REV: E

TECHNICAL DRAWING

BELIMED PREP & PACK WORKSTATION

TYPICAL EQUIPMENT LAYOUT OF A PREP & PACK
TABLE

DRAWN BY:

A. GRIFFIN

SCALE:

3/8" = 1'-0"

DATE:

10/27/2025

PAGE:

03 OF 05

WORKSTATION DIMENSIONS

A: WIDTH

- ☐ 48"
- ☐ 60"
- ☐ 72"

B: DEPTH

- ☐ 30"
- ☐ 36"

C: FRAME

- ☐ POWERED - (EL)
- ☐ ADJUSTABLE
- ☐ FIXED

D: SURFACE MATERIAL

- ☐ STAINLESS STEEL
- ☐ T-MOLD PLATIC LAMINATE (STANDARD COLOR GRAY)

E: OVERHEAD STORAGE

- ☐ 1. SHELF:
 - ☐ 8" X 30" ☐ 12" X 30" ☐ 15" X 30"
- ☐ QUANTITY: _____
- ☐ 2. 4" X 30" BIN RAIL
- ☐ 3. SPOOL HOLDER ROD
- ☐ 4. SMALL ROLL SHELF
- ☐ 5. PEEL PACK BOX DISPENSER (FOR BIN RAIL)

F: UNDER COUNTER STORAGE

- ☐ 1. SINGLE STACKING DRAWER:
 - ☐ 6" X 18" X 12" ☐ 12" X 18" X 12"
- ☐ 2. DUAL-STACKED DRAWERS 16" X 18" X 12"
- ☐ 3. DUAL-SLIDE OUT WIRE BASKETS

G: POWER MANAGEMENT

- ☐ 1. 8 OUTLET POWER BAR - (EL)
- ☐ 2. REAR WIRE TROUGH
- ☐ 3. POWER BAR WORK SURFACE MOUNT

H: PC SUPPORT

- ☐ 1. UNDER-SURFACE KEYBOARD & MOUSE TRAY
- ☐ 2. CPU UNDERMOUNT BRACKET
- ☐ 3. KEYBOARD ARM WITH MOUSE TRAY
- ☐ 4. HEIGHT-ADJUST ARM W/ KEYBOARD & MOUSE TRAY
- ☐ 5. FLAT PANEL DISPLAY ARM
- ☐ 6. HEIGHT-ADJUST FLAT SCREEN MONITOR ARM
- ☐ 7. COMBO MONITOR/ KEYBOARD ARM W/ MOUSE TRAY

I: OVERHEAD LIGHTING

- ☐ 48" DUAL-TUBE - (EL) ☐ 36" DUAL-TUBE - (EL)

J: TASK LIGHTING

- ☐ 1. FLUORESCENT CIRCULAR MAGNIFIER LAMP - (EL)
- ☐ 2. LARGE RING LED MAGNIFIER - 14W, 100-240V WITH CLAMP - (EL)
- ☐ 3. SMALL RING LED MAGNIFIER - 6W, 100-240V WITH CLAMP - (EL)
- ☐ 4. TASK LIGHT MOUNTING CLAMP

K: ORGANIZATIONAL AIDS

- ☐ 1. MAGNETIC MARKER BOARD - 24"x28"
- ☐ 2. PEG BOARD - 24"x28"
- ☐ 3. PULL-OUT WORK SURFACE - 11"x14"

L: CASTERS (ADJUSTABLE & POWER ONLY)

- ☐ 1. LEVELING CASTER
- ☐ 2. LOCKING CASTER
- ☐ 3. NON-LOCKING CASTER

M: MISCELLANEOUS

- ☐ 1. FREE STANDING FOOT REST - 14"x24"
- ☐ 2. COUNTER BACKSTOP
- ☐ 3. PROGRAMMABLE HEIGHT CONTROL
- ☐ 4. AIR TOOL KIT - (CA)

N: CARTS AND RACKS

- ☐ 1. SMALL SINGLE PRINTER STAND
- ☐ 2. LARGE SINGLE PRINTER STAND
- ☐ 3. LARGE DUAL PRINTER STAND
- ☐ 4. STERILE WRAP RACK - 60"x40"x18"

GENERAL NOTES

1. IN INSTANCES WHERE WORKSTATION(S) IS AGAINST WALL, UTILITY CONNECTIONS MAY BE INSTALLED THROUGH WALL.
2. FOR ILLUSTRATION CLARITY, NOT ALL AVAILABLE SELECTIONS ARE SHOWN.
3. SUGGESTED AREA(S) FOR UTILITIES PER TWO WORKSTATIONS.



8351 PALMETTO COMMERCE PKWY SUITE 101
LADSON, SC 29456
TEL: (800) 457-4117 WWW.BELIMED.COM

TDS-P&PW

REV: E

OPTIONAL ELEMENTS

BELIMED PREP & PACK WORKSTATION

BELIMED EQUIPMENT TECHNICAL SPECIFICATIONS

DRAWN BY:

A. GRIFFIN

SCALE:

3/8" = 1'-0"

DATE:

10/27/2025

PAGE:

04 OF 05

PREP & PACK WORKSTATION - TECHNICAL SPECIFICATIONS						
REQUIRED INPUTS						
Base Type	Description	Value	Utility Notes	Connection Method	Final Utility Conn. By:	
					Others	Belimed
CA (OPTIONAL)	Utility:	Compressed Air	12' Polyurethane retractable (coiled) hose with 1/4" ID provided by Belimed. Air must be dry and oil free. If configured with air tool kit.	Flex		X
	Connection Size/Type:	1/4 inch / MNPT				
	Pressure (Max):	30 psig				
EL (OPTIONAL)	Utility:	Electrical - Base	Powered base frame, includes 9.5 ft. line cord with NEMA 5-15 plug, center exit 2" above finished floor.	Receptacle		X
	Voltage:	120 V				
	Cycles:	60 Hz				
	Current/Watts:	1.6 A / 192W				
OPTIONAL OUTLET POWER BAR FOR ALL BASE TYPES						
	Utility:	Electrical	8 outlet power bar, includes 8 ft. line cord with NEMA 5-15 plug, right exit.			
	Voltage:	120 V				
	Cycles:	60 Hz				
	Current/Watts:	15 A / 1800W				
PHYSICAL SPECIFICATIONS						
FIXED	30" X 48":	160 lbs*	Maximum static load capacity 1000 lbs.			
	30" X 60":	189 lbs*				
	30" X 72":	220 lbs*				
	36" X 48":	177 lbs*				
	36" X 60":	215 lbs*				
	36" X 72":	257 lbs*				
PHYSICAL SPECIFICATIONS						
ADJUSTABLE	30" X 48":	192 lbs*	Maximum static load capacity 1000 lbs.			
	30" X 60":	228 lbs*				
	30" X 72":	261 lbs*				
	36" X 48":	207 lbs*				
	36" X 60":	252 lbs*				
	36" X 72":	287 lbs*				
PHYSICAL SPECIFICATIONS						
POWERED	30" X 48":	218 lbs*	Maximum frame lift capacity 500 lbs.			
	30" X 60":	254 lbs*				
	30" X 72":	291 lbs*				
	36" X 48":	223 lbs*				
	36" X 60":	272 lbs*				
	36" X 72":	314 lbs*				
	*Weights calculated with no accessories. Additional Information: Maximum height of 97" calculated at full frame extension, and equipped with optional overhead lighting system.					TS-P&PW_RevE